

WeCare Bed Cleaning Instructions

- Clean all electrical components (actuators, control box, pendant and cables) on your Joerns® bed with a damp, well-wrung cloth. Excess water in or on electrical parts can damage these components.
- Do not submerge the pendant in liquid.
- Your Joerns® bed has a durable, baked-on epoxy finish, easily cleaned with mild solution of antiseptic cleaner and warm water. Read and follow all label instructions on cleaning products.

Other Suggested Cleaners

- 3% hypochlorite or equivalent
 - Quaternary cleaners (active ingredient - ammonium chloride)
 - Phenolic cleaners (active ingredient - o-phenyl phenol)
 - Chlorinated bleach solution (5.25% - less than 1 part bleached to 100 parts water)
-
- Wood veneer surfaces should be maintained by cleaning with furniture polish or lemon oil. Laminate surfaces should be maintained by cleaning with disinfectant spray.

Warning: Possible Injury or Burns. Do not use flammable liquids or solvents to clean the bed, its components, or surfaces placed on top of it.

Caution: Possible damage to electrical parts, and/or shock hazard.